

January 14th 2019

Technical Information: 05AK18-EN

Introduction of new drain valves for autoclaves of Pro-Class (Vacuklav 23B+, 31B+) and S-Class (Euroklav 23S+ 23VS+, 29VS+)

Dear services partner,

As part of the continuous qualitative improvement of the Pro- and S-Class autoclaves, new storage tank drain valves were introduced.

Description of changes

In the past, there was a risk of clogging of the water filters in the drainage section. For this reason, we have converted the autoclaves mentioned above to the drain valves already proven in the autoclaves of the Premium Evolution class. The previous plug-in couplings are replaced and the water filters are no longer required.



Fig. 1: Connections for emptying the storage tank

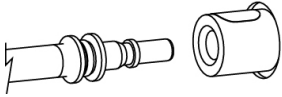

OLD	NEW
Plug-in coupling	Drain valve
	
The emptying hose engages noticeably.	Turn the emptying hose counterclockwise until it stops.

Fig. 2: Comparison of old and new design

Note: As part of this introduction, the front panel and the interior door mirror of the devices had to be adapted. It is therefore **not** possible to convert the old units to the new drain valves.

The new drain valves are factory-installed from the following serial numbers:

Device type	Serial number
Vacuklav 23 B+	201823-B2272
Vacuklav 31 B+	201831-B2186
Euroklav 23 S+	201823-S1311
Euroklav 23 VS+	201823V-S1843
Euroklav 29 VS+	201829V-S1371

The following new spare parts are available as of above serial numbers:

- Drain valve (1 piece) (Art. no.: 21815)
- Tank drain hose complete for Premium Evolution, Pro-Class, Euroklav (Art. no.: 53002)

For old devices, with plug-in couplings, the previous spare parts listed below can be ordered as usual.

- Plug-in couplings water drain (2 pcs) complete with mounting plate (Art. no.: 66690)
- Tank drain hose for Premium (41-B/43-B), Pro-Class, Euroklav (Art. no.: 53000)

Please let us know if you have any questions on the above topic.

Best regards

Customer Service

MELAG Medizintechnik oHG
Geneststr. 6-10
D-10829 Berlin

Tel.: +49 30 75 79 11 0
Fax: +49 30 75 79 11 99

E-Mail: hotline@melag.de
Web: <http://www.melag.com>